
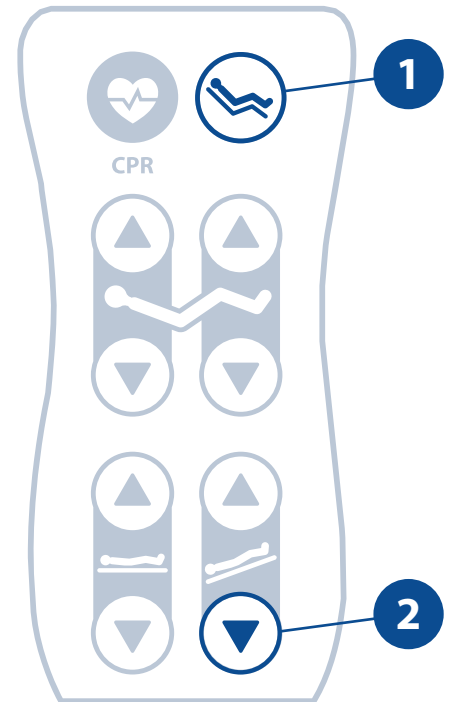


Sizewise Simplicity™: Cardiac Chair

Sizewise bed frames are easy to place in Cardiac Chair Position—with adjustable back, seat, and leg angles. To achieve the most clinically effective position for your patient, follow the two easy steps below.

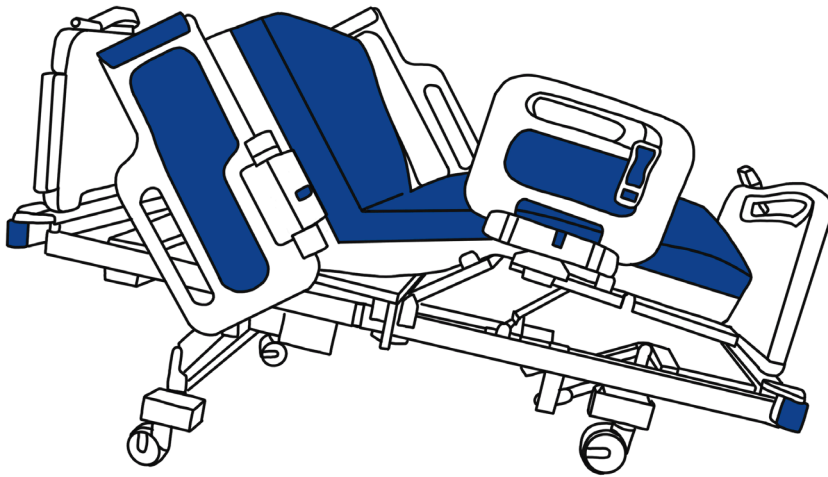

1. Press **Cardiac Chair** button
2. Adjust angle further using **Reverse Trend** button

Sizewise Bedframe	Cardiac Chair (max. angles)	
	Head	Knee
Bari Rehab Platform 2™	75°	43°
Bari Rehab Platform 3™	79°	20°
Evolution®	73°	34°
Evolution® Expandable	73°	34°
Low Boy®	73°	34°

Learn more: Call 800-814-9389
or contact your Sizewise rep

