


Pediatric Pulse™ Instructions

sizewise.com 800-814-9389

Lockout
Lockout will activate after three minutes of inactivity. To access control panel, press and hold for three seconds.

Alarms
Low Pressure Alarm
Press silence, then check power supply and all connections.
Power Failure Alarm
Press silence and check power supply.

Firm and Soft
To increase air flow: use Firm and Soft up and down arrows.

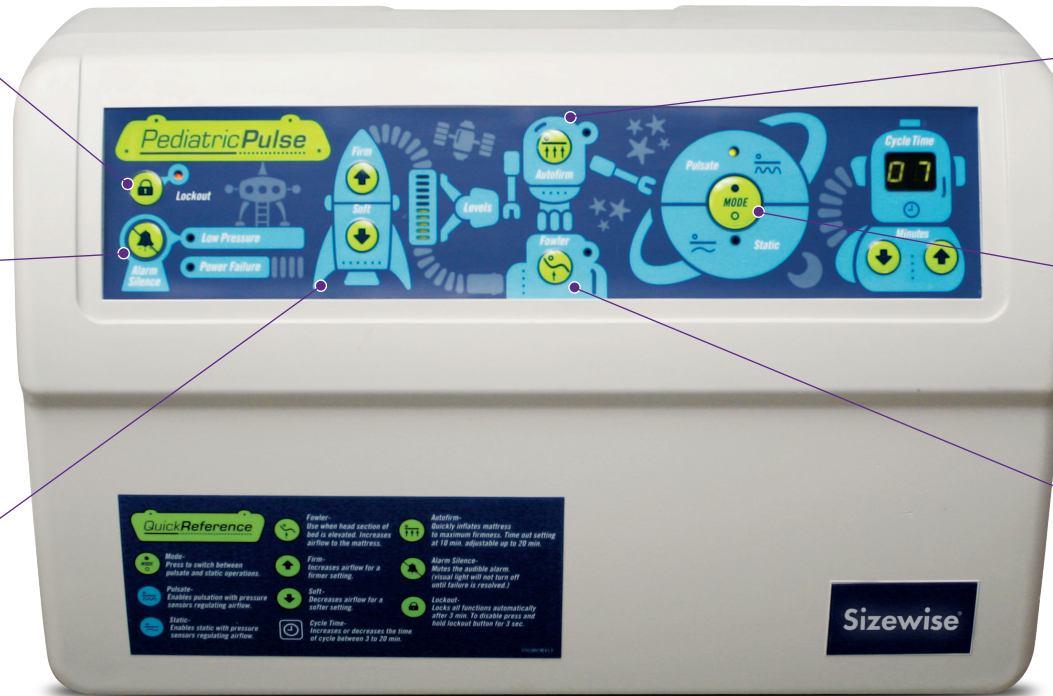

Autofirm
Press once for maximum inflation and a second time to remove. Autofirm will time out after 10 minutes and return to previous settings.

Mode
Press Mode button once for Static therapy and a second time for Pulsation. Adjust cycle times from 3-20 minutes using up and down arrows.

Fowler
Fowler feature will automatically start when head section is raised.

FOR CPR

- Turn off power switches on control unit
- Twist and open red CPR valve

FOR TRANSPORT

- Turn off power switches on control unit
- Unplug power cord from wall