

Behavioral Health Bed™

Quick Reference Guide



*Shown with optional
Rest Secure System™*

This card is not meant to replace product user manual. For complete manual, contact Sizewise.

800-814-9389
sizewise.com



Sizewise®
Designed to heal



Components



Side Rails



To lower rail: push button and rotate away from center of bed.

Note: Side rails are not removable for patient and caregiver safety.

Casters




Front and Rear casters lock and swivel

— To LOCK, press the lever down

— To UNLOCK, raise the lever

Removable Hand Control

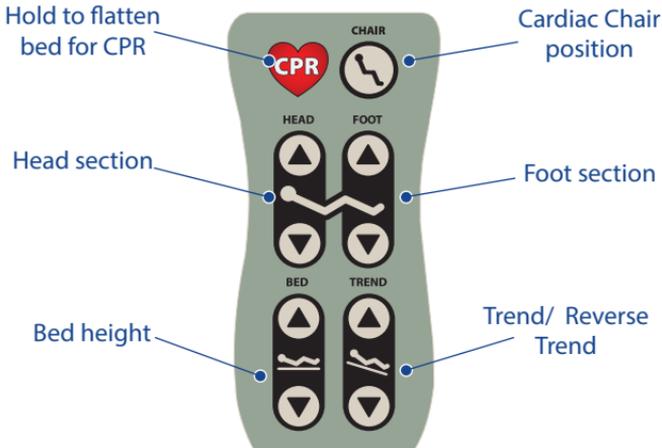


A removable hand control is provided to operate all bed functions.

Hand control can be attached to control box located near middle of bed, below mattress deck.

To disconnect hand control, squeeze end to release from control box.

Footboard Controls

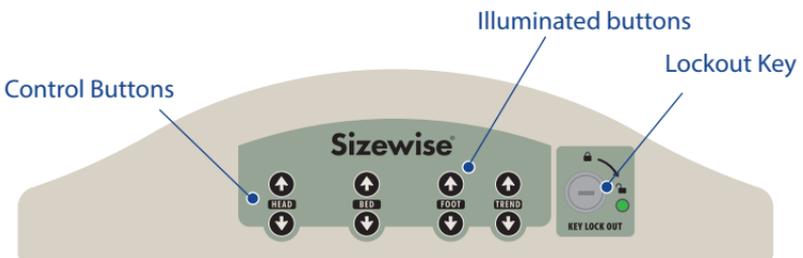

Located at foot end of bed, footboard controls have universal lockout key to enable/disable all functionality.

Lockout key disables footboard and removable hand control.

When LED is illuminated, controls are unlocked.

Battery Backup

This bed is equipped with an emergency battery backup that will charge when plugged into an AC outlet.

Troubleshooting

If bed fails to function:

1. Verify footboard control is unlocked
2. Verify power cord is plugged into working AC outlet and not cut, frayed, or damaged
3. Inspect hand control for damage and ensure it is properly connected to control box

If bed will not operate on battery power:

Verify hand control cable is properly connected to pendant connection port

For 24/7 support and service, call 800-814-9389.

Code Situation

1. Unlock casters and pull bed from wall.
2. Engage brakes and lower side rails for patient access
3. For LAL Support Surfaces: press CPR button or turn CPR dial on blower and/or open CPR valve at head of mattress if installed.
4. Press and hold CPR button on hand control to flatten bed. Footboard controls may also be used to flatten bed.
5. Proceed with facility protocols for resuscitation.