


Platinum 6000™ (Rev. 2)

Quick Reference Guide

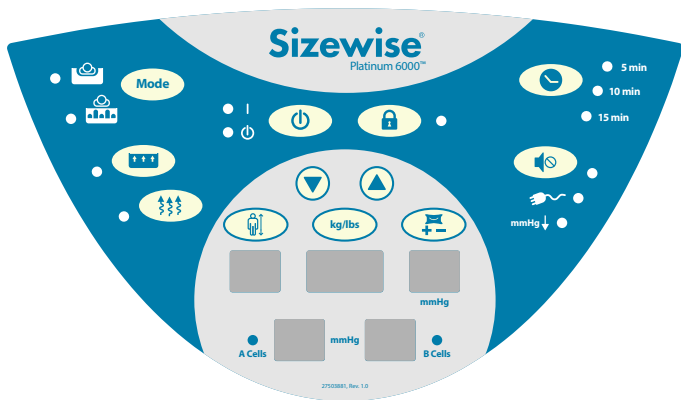

For Rev. 1 keypad, see Part No. 27650011


Sizewise®
Designed to heal

Keypad Controls



Power/Standby



Lock/Unlock



Mode: Select Immersion or Alternation



Immersion Therapy



Alternation Therapy



Autofirm: max inflate for transferring patient



Microclimate Management: on by default;
toggle to turn off



Cycle Time: for Alternation Therapy only



Alarm Silence: press to reset alarms





Power Fail Warning




mmHg ↓

Low Pressure Warning

Operating Instructions










1. Press and hold . Green LED indicates control unit is on; Amber LED indicates standby mode.
2. Surface should automatically enter Autofirm mode. If not, press  to max inflate surface.
3. Transfer patient to surface.

Setting Therapy Mode

1. Use  to select Immersion or Alternation.
—Alternation mode requires a 5, 10, or 15 min. cycle.
Use  to adjust.
2. Microclimate management is on by default.
To toggle off, press .

Patient Specific Adjustments





After setting Therapy Mode, customize the mattress for your patient by following three steps:

-  Patient Height: Select and use   until patient's height is shown on display.
-  Patient Weight: Select and use   until patient's weight is shown on display.
-  Comfort Settings: Select to override auto pressure settings. Use   to adjust surface's firmness (15-50 mmHg).

Switching Units of Measurement

The control unit's default settings are standard units.

Switch from Standard to Metric (or back):

1. Verify control unit is in standby or press  for two seconds to place into standby. Amber LED will show.
2. Press and hold  and  for five seconds.
3. Alarm will sound.
4. Press  to turn on.


Note: Decimal points in Patient Height or Patient Weight indicate metric units.

CPR/Quick Deflation

1. Locate red CPR valve at head of mattress. Turn valve for quick air release.
2. Detach hoses from control unit by pressing blue button and pulling connector from unit.

Patient Transport

To maintain inflation during transport:

1. Ensure CPR valve is in closed position.
2. Press  to put control unit into standby mode (amber LED).
3. If mattress is to be disconnected from control unit, place **Transport Cap** over hose connections.

This card is not meant to replace product user manual. For complete manual, contact Sizewise.

sizewise.com 800-814-9389

OPS-0047B-1118 27650013, Rev. 1.0

